LIVING

CARE CONCEPT

 **SMEDBO**
Life in your bathroom

LIVING - a flexible care concept by Smedbo®

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EN LIVING is an elegant bathroom series with focus on flexibility and safety in the bathroom.

The series is characterized by quality and comfort. LIVING bathroom handles are designed in modular systems and can be customized to suit the needs and appearance of the bathroom. The component parts of the modular systems can be replaced with free-standing parts, for example other lengths. The module systems can also be built together. Together, the modular systems and the freestanding parts offer almost unlimited design possibilities.

OBSERVE

- Please read the instructions carefully before installation and use.
- Installation should be carried out by a qualified professional.
- Take into account current wet room recommendations and standards when installing.
- Before mounting, check the suitability and load-bearing capacity of the wall/substrate. Note that the wall/base is the limiting factor at high load. The wall/base must withstand a maximum load of 150kg.
- The bathroom handles are not recommended to be installed on wetroom wall paper.
- Supplied screw are recommended only for cement wall. For other surfaces, choose screws based on the wall material and construction.
- For horizontal combinations, maximum distance allowed between brackets are 600 mm.
- The 1100 mm long part of the FK814/FK816 may only be mounted vertically. It is not approved for horizontal level placement.
- Check the screws every 12 months and tighten if necessary.

TEST

Maximum load 150kg
Tested according to SS-ISO
17966:2016, 23.7.3
Tests performed by RISE.*

**Research Institutes of Sweden*

WARNING

- Exceeding the maximum load can lead to injury and/or damage to property.
- Failure to follow instructions may result in bodily injury.

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SMEDBO LTD
SMEDBO NORGE A/S